

# **MINT-A-KLEEN ASSISTANT**®

## **Suction Funnel for Expelled Waterline Waste**

[Dental Unit Waterline Cleaning Accessory for MINT-A-KLEEN®]

Item #: MAK-1380

---

### **INDICATIONS FOR USE:**

MINT-A-KLEEN ASSISTANT® is a waterline cleaning accessory, namely, a suction funnel for expelled dental waterline waste and dental water that is used during Mint-A-Kleen's Waterline Cleaning Treatment.

### **FEATURES & BENEFITS:**

- **SAVES TIME:** The top of MINT-A-KLEEN ASSISTANT is designed to hold 4 dental waterlines (2 handpiece hoses and 2 three way syringes) simultaneously when purging or flushing waterlines during Mint-A-Kleen.
- **EASY INSTALLATION:** MINT-A-KLEEN ASSISTANT's base is inserted into the high-speed suction.
- **COLLECTS CONTAMINATION:** The waterline waste that is expelled when purging and/or flushing the dental waterlines is contained and collected by MINT-A-KLEEN ASSISTANT.
- **WATERLINE WASTE REMOVED BY SUCTION:** The waterline waste and dental unit water from the dental waterlines expels into the MINT-A-KLEEN ASSISTANT and is then removed by suction.
- This method **LESSENS** possible aerosol contamination and splatter from expelled waterline contamination.
- **PROTECTS** dental staff from the aerosol contamination and splatter.
- MINT-A-KLEEN ASSISTANT serves as a **VISUAL REMINDER** to the dental staff and dentist that Mint-A-Kleen still remains inside the dental unit.
  - MINT-A-KLEEN ASSISTANT should be connected during the entire Mint-A-Kleen® waterline treatment.

### **BEFORE YOU START**

Before adding Mint-A-Kleen® to the dental unit waterlines, the dental unit water in the handpiece hoses and three way syringes must be expelled. Mint-A-Kleen's 1<sup>st</sup> Step, "AIR PURGE", removes the water from the waterlines. This dental unit water is expelled into the MINT-A-KLEEN ASSISTANT, which is removed by the high-speed suction.

Mint-A-Kleen's Final Step, "FLUSH DENTAL WATERLINES WITH WATER", removes contamination, Mint-A-Kleen, dental unit water and other debris into the MINT-A-KLEEN ASSISTANT, which is removed by the high-speed suction.

---

### **DIRECTIONS FOR USE:**

1. Insert MINT-A-KLEEN ASSISTANT's base into the dental unit's high-speed suction.
2. Turn high speed suction **ON**.
3. Insert handpiece hose ends into openings on top of the MINT-A-KLEEN ASSISTANT. Activate foot control.
4. Place syringe tip in the opening of the MINT-A-KLEEN ASSISTANT and activate syringe's water button.
5. **NOTE:** Mint-A-Kleen's 1<sup>ST</sup> Step: AIR PURGE is completed when water is no longer exiting syringe tips and handpiece hoses. Mint-A-Kleen's Final Step: FLUSH WATERLINES WITH WATER is completed after flushing the dental waterlines with water for 30 seconds.
6. Remove MINT-A-KLEEN ASSISTANT when Mint-A-Kleen's dental waterline cleaning protocol is completed.
7. Rinse MINT-A-KLEEN ASSISTANT. Do not autoclave.