

WaterlineX®

CLOSED Dental Water Bottle System With Air Purge

Anodia Systems Dental Manufacturer #: MAK-1806

INSTALLATION INSTRUCTIONS

STEP 1 | **MOUNTING WaterlineX**

- A. Identify a suitable location to mount on a vertical 2" post or a vertical flat surface.
- B. WaterlineX should be mounted in a suitable location so the dentist or dental staff has easy access.
- C. The location should be as close as possible to the dental unit's internal tubing of air and water.

TUBING CONNECTIONS STEP 2 |

- A. Turn dental unit **OFF**.
- B. Locate a 1/8" Air line (yellow) connected to the dental unit's ON/OFF switch. Cut this line and "T" to line to the WaterlineX Air line.
- C. Connect dental unit's waterline.

STEP 3 **INSTALLATION CHECK**

A. Turn dental unit **ON** and check for air leaks.

STEP 4 **OPERATION CHECK**

- A. Turn WaterlineX System OFF.
- B. Fill dental water bottle with water.
- C. Turn **WaterlineX** System **ON** and check for water flow and leaks.



