

DentaLINE[®] Connector

Dental Waterline Cleaning Accessory by Anodia Systems, LLC

Item #: MAK-1277

Designed for use with EPA Registered Dental Waterline Cleaners by Anodia Systems, LLC:

- **SHOCKING[®]** (Item #: MAK-SHOCK)
- **MINT-A-KLEEN[®]** (Item #: MAK-OZ)

USER: DentaLINE is for MINT-A-KLEEN[®] or SHOCKING[®] Waterline Cleaner Users that have a Water Purifying Filter Cartridge installed in their Dental Water Bottle's Pick-Up Tube. Dental Water Bottles that have Water Purifying Filter Cartridges should Remove BEFORE Waterline Cleaning Treatment. Do not Filter MINT-A-KLEEN[®] or SHOCKING[®].

INDICATIONS FOR USE: DentaLINE Connector is an Elongated Pick-Up Tube that Prevents a Waterline Cleaner from being Filtered. Replaces the Water Purifying Filter Cartridge During Dental Unit Waterline Cleaning Treatment. DentaLINE Connects where the Cartridge was attached to the Original Dental Unit Manifold's Waterline Pick-Up Tube.

DIRECTIONS FOR USE:

*****Note:** Install DentaLINE Connector BEFORE MINT-A-KLEEN Waterline Cleaning Treatment.

1. Disconnect the Water Filter Cartridge from the Dental Unit Manifold's Pick-Up Tube.
 - a. DO NOT Filter MINT-A-KLEEN through Cartridge.
2. Keep the Cartridge's Clear Plastic Connector on the Dental Unit Manifold's Pick-Up Tube (See Picture 2 Below).
3. Insert our DentaLINE Connector Pick-Up Tube into the Cartridge's Clear Plastic Connector.
 - a. Hold the Cartridge's Clear Female Connector.
 - b. **Twist & Push** our DentaLINE Male Connector into the Cartridge's Clear Female Connector.
4. Place our DentaLINE's Tube into the Dental Water Bottle.

NOTE*** If DentaLINE tube is too long for the bottle: Cut the length (the end going into the bottle) at a 45-degree angle. The end of the DentaLINE tube should contact the bottom of the dental water bottle.

5. After MINT-A-KLEEN's Cleaning Treatment is Completed, the User will *Disconnect* DentaLINE.
 - a. Rinse DentaLINE with Water for future MINT-A-KLEEN Dental Waterline Cleaning Treatments.
6. Re-Connect Water Purifying Filter Cartridge to the Original Dental Unit Manifold's Waterline Pick-Up Tube.
7. Begin Dental Procedures.

