

# DentaKLEEN®

## WATERLINE CLEANING SYSTEM FOR DENTAL UNITS ON CITY WATER

ITEM #: MAK-1900

### **INSTALLATION GUIDE**

#### **STEP 1: PREPARE FOR INSTALLATION**

1. Turn Dental Unit **OFF**.
2. Confirm that no Air or Water comes out of the 3-Way Syringe.

#### **STEP 2: MOUNTING DentaKLEEN® SYSTEM**

1. Identify a Location to Mount on a Vertical 2" Light Post or a Flat Surface.
2. The Location should be as close as possible to the Dental Unit's Internal Tubing of Air and Water Supply.

#### **STEP 3: MAKING THE CONNECTIONS**


1. On the Dental Unit, Locate a 1/8" Air Line (Yellow) Connected to the Dental Unit's ON/OFF Switch.
2. Confirm the ON/OFF Switch is a 3-Way Switch.
3. Cut this Yellow Air Line and "T" to the DentaKLEEN® SYSTEM's Air Line.
4. On the Dental Unit, Locate a 1/4" Waterline (Blue); Cut this Blue Waterline and "T" the DentaKLEEN® SYSTEM to this Line.

#### **STEP 4: OPERATION CHECK**

1. Turn Water and Air **ON** at Dental Unit's Junction Utility Box.
2. Turn Dental Unit **ON** and Check for Air and Water Leaks.

# DentaKLEEN®

## INSTALLATION GUIDE

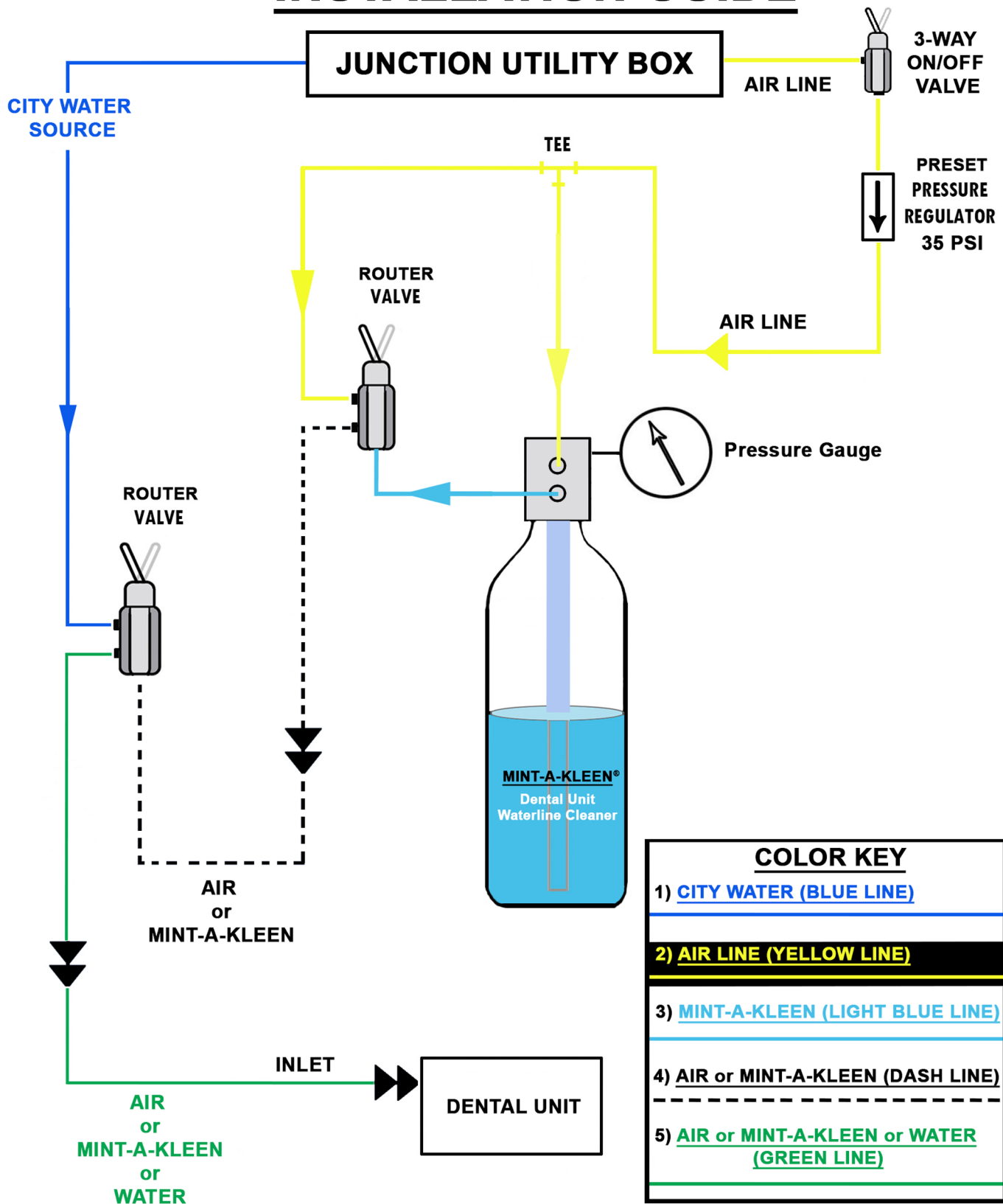

### COLOR KEY

- 1) CITY WATER (BLUE LINE)
- 2) AIR LINE (YELLOW LINE)
- 3) MINT-A-KLEEN (LIGHT BLUE LINE)
- 4) AIR or MINT-A-KLEEN (DASH LINE)
- 5) AIR or MINT-A-KLEEN or WATER (GREEN LINE)