

VELOCITY WATER®

CITY WATER DELIVERY SYSTEM

ITEM #: MAK-1805

INSTALLATION GUIDE

STEP 1: PREPARE FOR INSTALLATION

1. Turn Dental Unit **OFF**.
2. Confirm that no Air or Water comes out of the 3-Way Syringe.

STEP 2: MOUNTING VELOCITY WATER® SYSTEM

1. Identify a Location to Mount on a Vertical 2" Light Post or a Flat Surface.
2. The Location should be as close as possible to the Dental Unit's Internal Tubing of Air and Water Supply.

STEP 3: MAKING THE CONNECTIONS

1. On the Dental Unit, Locate a 1/8" Air Line (Yellow) Connected to the Dental Unit's ON/OFF Switch.
2. Confirm the ON/OFF Switch is a 3-Way Switch.
3. Cut this Yellow Air Line and "T" to the VELOCITY WATER® SYSTEM's Air Line.
4. On the Dental Unit, Locate a 1/4" Waterline (Blue); Cut this Blue Waterline and "T" the VELOCITY WATER® SYSTEM to this Line.

STEP 4: OPERATION CHECK

1. Turn Water and Air **ON** at Dental Unit's Junction Utility Box.
2. Turn Dental Unit **ON** and Check for Air and Water Leaks.

VELOCITY WATER®

INSTALLATION GUIDE

