

VELOCITY WATER®

CITY WATER DELIVERY SYSTEM

ITEM #: MAK-1805

INSTRUCTIONS

INDICATIONS FOR USE:

With the Flip of a Toggle Switch – Clean your Dental Waterlines while Remaining on City Water.

VELOCITY WATER is Designed to Clean Dental Unit Waterlines for Dental Units Connected to City Water without a Bottle System. **VELOCITY WATER** is a City Water Delivery Bottle System that uses 3 Toggle Switches. **Toggle Switch #1:** Master ON/OFF Switch **Toggle Switch #2:** Controls City Water Flowing into Dental Unit. **Toggle Switch #3:** Air Purges Dental Unit / Adds Mint-A-Kleen or Preferred Dental Unit Water Source into Dental Unit.

SIZE: 1 Liter BOTTLE SYSTEM

GENERAL DENTAL PROCEDURES:

***Please Read and Follow **VELOCITY WATER**'s Installation Instructions Prior to Operation.

- 1). After Installation is Completed, Turn City Water's Toggle Switch to **ON**.
- 2). Begin Dental Procedures.

VELOCITY WATER's Dental Waterline Cleaning Directions using MINT-A-KLEEN®

1. Turn Dental Unit **OFF** and **VELOCITY WATER**'s Master Switch **OFF**.
2. Remove **VELOCITY WATER**'s Bottle.
3. Fill the Empty Bottle with MINT-A-KLEEN®.
4. Attach **VELOCITY WATER**'s Bottle Filled with MINT-A-KLEEN® onto the DentaKLEEN's System.
5. Turn Dental Unit **ON** and **VELOCITY WATER**'s Master Switch **ON**.
6. Turn **CITY WATER** Toggle Switch to **OFF**.
7. Turn Toggle Switch to **AIR**.
8. *Air Purge* Dental Unit Waterlines by Activating Handpiece Hoses and Air/Water Syringes until Dental Unit Water is Eliminated.
9. Turn Toggle Switch to **MINT-A-KLEEN®**. *Activate* Handpiece Hoses and Air/Water Syringes until MINT-A-KLEEN® Exits.
10. Turn Dental Unit **OFF** and **VELOCITY WATER**'s Master Switch **OFF**. Allow MINT-A-KLEEN® to Treat Waterlines for at least 12 Hours (Preferably Over the Weekend Before Proceeding to the Next Step).
11. Turn Dental Unit **ON** and **VELOCITY WATER**'s Master Switch **ON**.
12. Confirm **CITY WATER** Toggle Switch is **OFF**.
13. Turn Toggle Switch to **AIR**.
14. *Air Purge* Dental Unit Waterlines by Activating Handpiece Hoses and Air/Water Syringes until MINT-A-KLEEN® is Eliminated.
15. Turn **CITY WATER** TOGGLE SWITCH to **ON**.
16. *Flush* Waterlines with Water for 30 Seconds to Remove Any Remaining Waterline Waste.
17. Begin Dental Procedures.

* **NOTE** – For Optimal Cleaning, Air Purge and Clean All Instruments and Tubing that Contain Water using MINT-A-KLEEN® and its Cleaning Protocol. Please use the DEAD ENDS Drain Tube to Drain & Remove the Dental Unit's Water. Water Heaters Must Be Cleaned As They Increase the Amount of Bacteria in the Dental Waterlines. If a Tank Type Water Heater is in Use, Please Consider the Following Important Information. Most Water Heaters of this Style Cannot Be Run Dry When Turned On. They Must Be Shut Off Prior to Being Drained During Air Purging. Also After the Use of Mint-A-Kleen or Any Liquid Dental Unit Waterline Cleaner, the Unit Must be Air Purged First to Remove the Solution from the Water Heater and Then Flushed with Water for 30 Seconds. Remember Water Heaters Must Be Turned ON After the Dental Unit Waterlines Are Cleaned. FAILURE TO TURN TANK TYPE WATER HEATERS OFF DURING THE PROCESS OF CLEANING THE WATERLINES COULD CAUSE THE WATER TANK ELEMENT TO FAIL.



Anodia Systems, LLC, Dental Manufacturer
109 Larrimore Lane | Danville, KY 40422 USA
P: 866.246.2548 | E: info@AnodiaSystems.com
www.AnodiaSystems.com