

ITEM #: MAK-1910

PRO-KLEEN® INSTRUCTIONS



INDICATIONS FOR USE

PRO-KLEEN® SYSTEM is a Portable, Pressurized Dental Unit Waterline Cleaning System Designed to Dispense Mint-A-Kleen®, Dental Unit Waterline Cleaner, to Dental Units without Bottle Systems that are Connected to City Water. Now offices on City Water without Bottle Systems Do Not have to Invest in Bottle Systems in order to Treat and Maintain Clean Dental Unit Waterlines. Administering Mint-A-Kleen using PRO-KLEEN is a Safe, Affordable, Convenient and Effective Cleaning System for Offices on City Water that are Committed to Dental Unit Waterline Infection Control.

DIRECTIONS FOR USE

IMPORTANT

- Please Read and Follow the **PRO-KLEEN Installation Guide** Prior to Operation.
- Read Mint-A-Kleen®, Dental Unit Waterline Cleaning Instructions Prior to using PRO-KLEEN®.
- Check Mint-A-Kleen Fluid Level in PRO-KLEEN and Replace/Refill Bottle if necessary.
- Ensure Dental Unit's Master Toggle Switch is in the **'ON'** Position.
- Prior to Dental Unit Waterline Cleaning Treatment, Remove Handpieces, Scaler Insert and Syringe Tips.
- Notice Regarding Low-Speed Handpieces: If the Dental Unit's Low-Speed Handpiece is Dry, Ensure that the Handpiece Tubing Waterline is Disconnected from the Control Block and that the appropriate Barb Fitting is Removed and Replaced with the provided Plug.

HOW TO USE PRO-KLEEN® WITH MINT-A-KLEEN®

1. Attach a Full Mint-A-Kleen Bottle to PRO-KLEEN's Manifold.
2. Turn the Dental Unit's Utility Box's Toggle Switch to the **OFF** Position.
3. Insert PRO-KLEEN's **YELLOW** and **BLUE** Male Quick Connects into like colored Female Quick Connects on Dental Unit's Utility Box. **YELLOW** is 'AIR SUPPLY' and **BLUE** is 'FLUID CONNECTION'.
4. Turn PRO-KLEEN Toggle Switch to the **AIR PURGE** Position.
5. Turn Dental Unit's Utility Box's Toggle Switch to the **ON** Position and Run Handpieces, Syringe, Scaler and any other Oral Cavity Water-Consuming Device until only Air is Expelled.
6. Turn the PRO-KLEEN Toggle Switch to the **Mint-A-Kleen** Position and Run Handpieces, Syringe, and Scaler any other Oral Cavity Water-Consuming Device until Mint-A-Kleen is Visible.
7. Remove PRO-KLEEN's **YELLOW** and **BLUE** Quick Connects from the Dental Unit's Utility Box.
8. Turn Dental Unit's Utility Box's Toggle Switch to the **OFF** Position.
9. **DO NOT** use Dental Unit for Dental Procedures when Mint-A-Kleen is Inside Treating the Dental Unit Waterlines.
10. Allow Mint-A-Kleen to Treat Dental Unit Waterlines for at Least 12 hours before Flushing with Water.
11. **FLUSH DENTAL WATERLINES AFTER A MINIMUM OF 12 HOUR MINT-A-KLEEN TREATMENT**
12. FLUSH Handpieces, Syringe, Scaler and any other Oral Cavity Water-Consuming Device with WATER for 30 Seconds.
13. Begin Dental Procedures.

IMPORTANT NOTICE REGARDING WATER HEATERS

If a tank type water heater is in use, the following important information must be considered. Most water heaters of this style cannot be run dry when turned on. Water Heaters must be shut **OFF** prior to being drained. As both the PRO-KLEEN and bottle systems purge air through the waterlines prior to injecting MINT-A-KLEEN liquid, it is critical that the water heater be turned OFF during this process.

FAILURE TO TURN TANK TYPE WATER HEATERS OFF DURING THE PROCESS OF ADMINISTERING MINT-A-KLEEN COULD CAUSE WATER TANK ELEMENT FAILURE.