

# DentaLOCK®

## QUICK CONNECT BOTTLE ADAPTER KIT

BY ANODIA SYSTEMS  
ITEM #: MAK-1861

### INDICATIONS FOR USE:

Save time changing your bottle with the **DentaLOCK** Bottle Adapter. DentaLOCK is designed to retrofit your existing dental unit water bottle system with standard 28 mm bottle threads. DentaLOCK prevents cross threading by eliminating the need to screw a bottle on and off the bottle system.

### IMPORTANT

Depressurize dental water bottle before removal. Turn **OFF** the dental unit water bottle system or dental unit's Master Switch to relieve bottle pressure prior to bottle removal.

### INSTALLATION INSTRUCTIONS

- 1). Turn dental unit water bottle system **OFF** to relieve bottle pressure.
- 2). Unscrew the dental water bottle from the dental unit's manifold.
- 3). Screw the **Bottle Adapter (1)** tightly onto dental water bottle.
- 4). Screw the **Black Lock-Ring (2)** tightly onto the **Bottle Adapter (1)**, which is now attached to the dental water bottle.
- 5). Attach the new pick-up tube into the dental unit manifold.
- 6). Guide pick-up tube through the **Dental Unit Manifold Adapter (3)**
- 7). Screw the **Dental Unit Manifold Adapter (3)** into the dental unit's manifold. Secure tightly.

### INSTRUCTIONS FOR BOTTLE ATTACHMENT

**STEP 1:** Connect your dental bottle with the attached **Bottle Adapter (1)** and **Black Lock-Ring (2)** to the **Dental Unit Manifold Adapter (3)** by grasping the upper **Black Lock-Ring (2)** with one hand and Pull Down on it while pushing the **Dental Unit Manifold Adapter (3)** that is attached to dental unit securely into the **Black Lock-Ring (2)** with the other hand.

**STEP 2:** Release **Black Lock-Ring (2)** to securely connect to **Dental Unit Manifold Adapter (3)**.

### INSTRUCTIONS FOR BOTTLE REMOVAL

**STEP 1:** To Remove bottle, turn dental unit **OFF** to release pressure.

**STEP 2:** Simply grasp the **Dental Unit Manifold Adapter (3)** and with the other hand gently Pull Down the Upper Part of the **Black Lock-Ring (2)** to slowly remove the Bottle in a downward motion.

**NOTE:** If upper **Black Lock-Ring (2)** does not easily move, then pressure is still inside dental water bottle.

Turn dental unit's Master Switch **OFF**, then remove bottle.

